


GeeWhiz® Condom Catheter

Instructions for use

Note: We recommend downloading these instructions as a printable Adobe Acrobat PDF file by [clicking here](#). Download & Print @ 100%

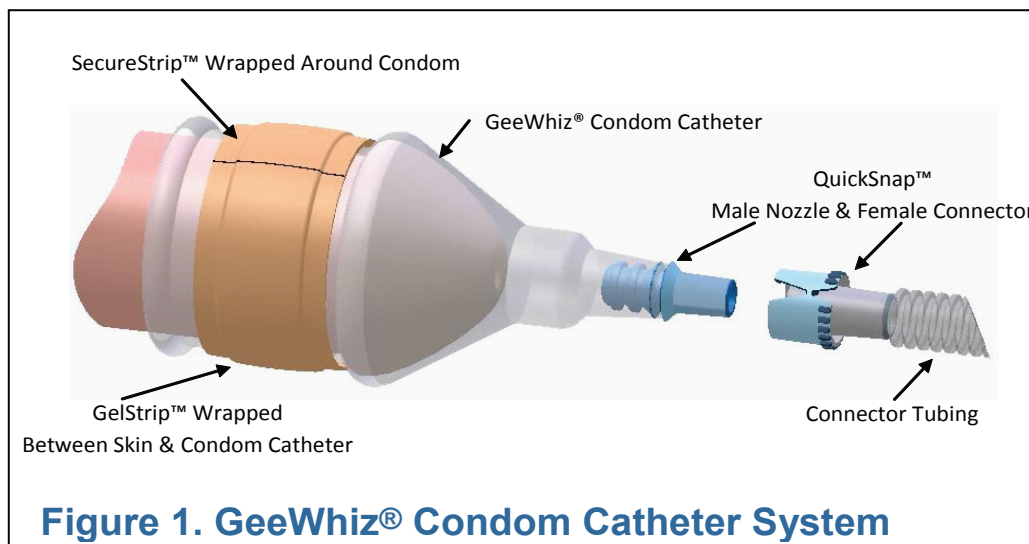

Selecting The Correct Size Of The GeeWhiz Condom Catheter

Prior to using the GeeWhiz Condom Catheter, select the “CORRECT SIZE” to best fit your needs. Available condom sizes are: 29 MM, 32 MM, 36 MM and 41 MM. Use the GeeWhiz Condom Sizer™ template and follow instructions for best results. Make a record of your size for future reference and ordering.

NOTE: You may download, print, cut and use the Sizing Guide shown below.

Download Chart at 100%. Or, if you prefer, call us and we will gladly mail you a full size Sizing Guide at no charge.

Sizing Guide


Cut along dotted lines. Place openings over penile shaft behind the glans or head of penis to select correct size.

29 MM
(1.14 Inches)

32 MM
(1.26 Inches)

36 MM
(1.42 Inches)

29 MM
(1.61 Inches)

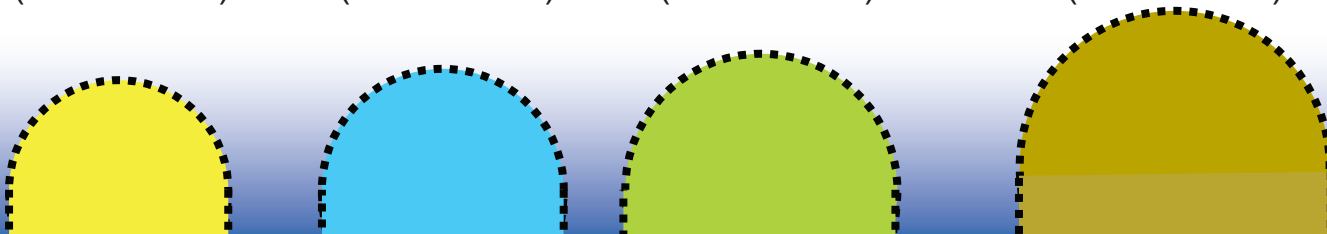


CUT OUT STRIP. WRAP AROUND SHAFT OF PENIS. USE SIZE CLOSEST TO ARROW TIP



TO APPLY THE GEEWHIZ CONDOM CATHETER YOU WILL NEED THE FOLLOWING ITEMS:

One GeeWhiz Daily Pak™ containing:

- 1-GeeWhiz® Condom Catheter with male QuickSnap™ nozzle in tip
 - 1-GelStrip™ silicone adhesive strip
 - 1-SecureStrip™ Wrapping Tape
 - 1 Length of connector tubing
- Leg Bag or Bedside Drain Bag (not Shown)

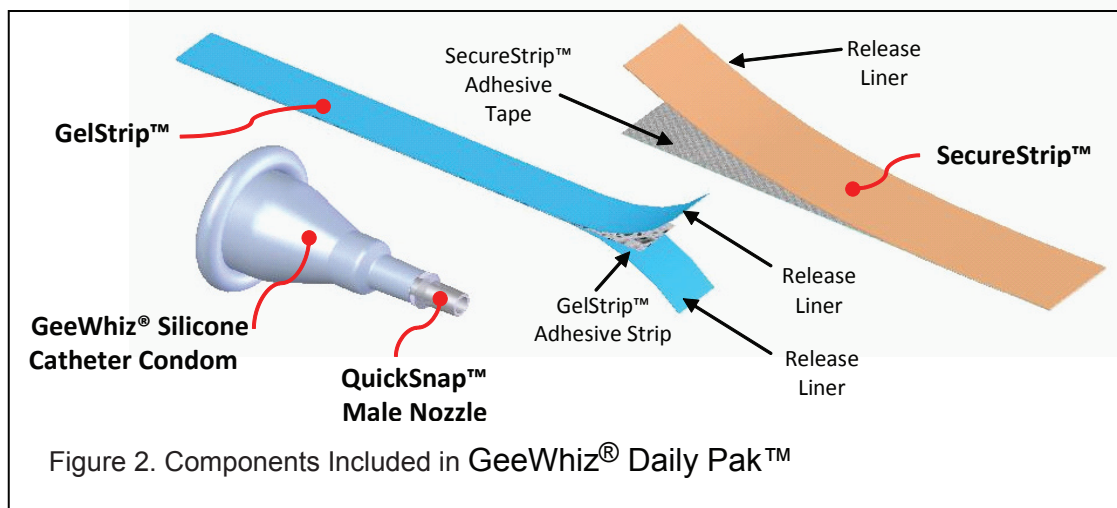

It's As Simple As 1, 2, 3!

Quick Reference Guide for Applying the **GeeWhiz®** Condom Catheter
Refer to Instruction Booklet & Read Complete Instructions Before Using Product


1. Remove both plastic liners from the GelStrip™ adhesive gel and place the GelStrip on the surface of the penis behind the glans. While applying a slight tension to the GelStrip, wrap it completely around the outside of the penis. Overlap the GelStrip and press it into place around the penis – **DO NOT CUT!**

NOTE: Application Method Differs With The Non-Circumcised Male.
The Foreskin Must First Be Pulled Forward Over The Glans Of The Penis Before The GelStrip Is Wrapped Around The Penis Behind The Glans!

2. Place the head of the penis into open end of condom. Unroll the condom onto the penis, over, and all the way across the GelStrip until the rolled end of the condom is behind the edge of the GelStrip nearest your body. Squeeze the condom around the circumference of the GelStrip to insure complete contact and a good seal.

3. Remove the protective liner from the SecureStrip.™ Center the tape on the outside surface of the condom over the GelStrip "Bump." While applying a slight tension to the tape, wrap it completely around the outside of the condom – **DO NOT CUT!**

CAUTION: Do not wrap so tightly as to restrict the flow of urine or blood.

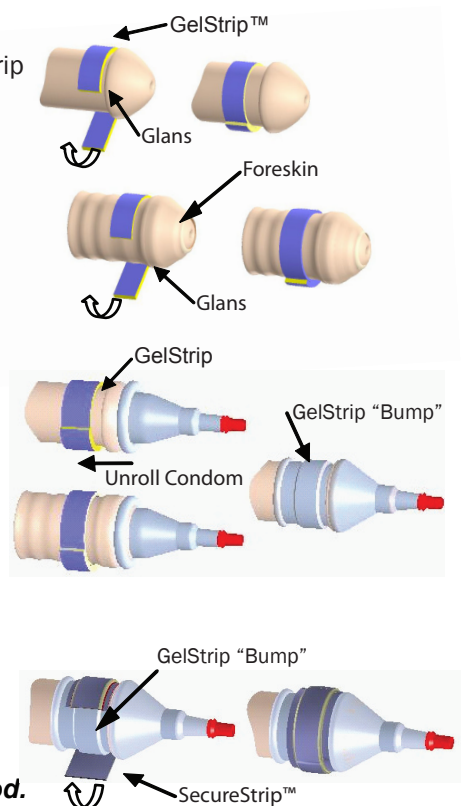

IT'S MEDICARE APPROVED-DIAPERS ARE NOT!

MANUFACTURED BY
LEI LEADING EDGE INNOVATIONS, INC.
"TOMORROW'S PRODUCTS TODAY"
CAMARILLO, CA