



www.alphadry.com  
1-877-235-9379

Warning: This product is made from latex which can cause allergic reactions in some individuals.

# AlphaDry Instructions

## ALPHADRY INCLUDES:

- ❖ AlphaDry Condom/Reservoir
- ❖ Velcro Strap
- ❖ Instructions/Care/Troubleshooting

Note: Use brief-style underwear.



## Step 1



### INITIAL PREPARATION:

Twist to open blue valve. Make sure penis and inside of condom are dry.

## Step 2



### ROLLING CONDOM DOWN:

Place thumbs over the inside bead and forefingers on the outside bead and begin to roll down like a condom.

## Step 3



### ROLL CONDOM DOWN TO INTERNAL VALVES:

Roll condom down until the internal valves are clearly showing.

## Step 4



### POSITION TO ROLL ALPHADRY ON PENIS:

Position condom at the head of penis as shown, with fingers of both hands on the bead and thumbs on the head of penis.

## Step 5

### BEGIN TO ROLL CONDOM ONTO PENIS:

With your fingers, begin to roll the condom over the head of the penis.



Note: Make sure AlphaDry is the correct size. It should be snug when rolling it onto the penis. If it is too loose or too tight, you may order one AlphaDry replacement of a different size, at no extra charge, by calling 1-877-235-9379.

## Step 6

### PULL FORESKIN BACK:

With one hand pull foreskin back as far as it is comfortable and with the other hand continue to roll the condom on.



Note: Make sure head of penis is up against the valves and foreskin is pulled back. Reposition if necessary.

## Step 7

### UNROLL CONDOM TO BASE:

Unroll condom with both hands to base of penis.



**More Instructions** ➔



www.alphadry.com  
1-877-235-9379

**Warning:** This product is made from latex which can cause allergic reactions in some individuals.

# AlphaDry Instructions

8082 5M 3.07


## Step 8



### WRAP ELASTIC STRAP BEHIND HEAD OF PENIS:

Hold the soft edge end of the strap about 1/2 inch away from the head of the penis angled slightly as shown. Begin to wrap the remainder of the strap right against the head of the penis with tension.

## Step 9

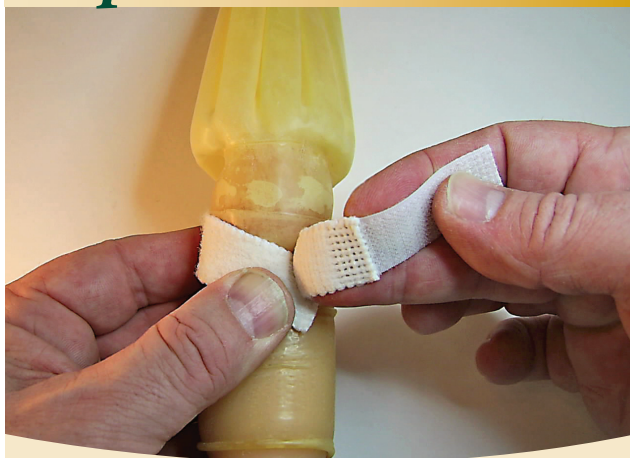

### LOCK STRAP DOWN:

After the strap is stretched, wrap strap on around and lock it down.

## Step 10

### IN PLACE ELASTIC STRAP

This is the correct position of the elastic strap.



**Note:** Positioning the strap just behind the head of the penis with tension is the key to preventing slipping and leaking. Adjust tension for comfort and to maintain dryness.

## Step 11



### POSITION IN UNDERWEAR

Lay AlphaDry positioned somewhat horizontal, left or right in your underwear. The Blue Valve will be almost to your side below the elastic in your underwear. Think of a banana lying horizontally in your underwear.

**Note:** This position allows the one-way valves to be free and open. Do not make a sharp bend in your AlphaDry.

## Step 12



### CLOSE BLUE VALVE

Close the blue valve keeping the AlphaDry in position. You are now ready to experience dryness in a comfortable way and resume normal activities.

**Remember:** Replace AlphaDry every 15 days of use for best results and to minimize risk of infection.



You are now ready to resume life's normal activities with greater confidence.

### SEE INSERT CARD FOR:

- ❖ Emptying the AlphaDry
- ❖ Troubleshooting
- ❖ Removing the AlphaDry
- ❖ Care and Helpful Hints